

# USER MANUAL



Thank you for choosing Better Bathrooms.  
Please read this manual before installing your product  
and keep for future reference.

**better  
bathrooms**


Smooth Grab Rail 350x77mm

Smooth Grab Rail 450x77mm

Smooth Grab Rail 600x77mm

Smooth Grab Rail 700x77mm

Knurled and smooth Grab Rail 800x77mm

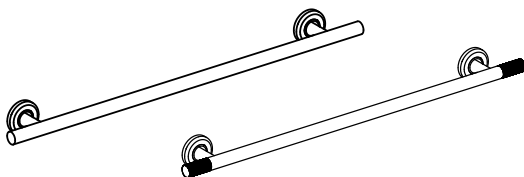

ARIMB350GR

ARIBB350GR

ARIGM350GR

ARIBR350GR

ARIMB450GR

ARIBB450GR

ARIGM450GR

ARIBR450GR

ARIMB600GR

ARIBB600GR

ARIGM600GR

ARIBR600GR

ARIMB700GR

ARIBB700GR

ARIGM700GR

ARIBR700GR

ARIMB800GR

ARIBB800GR

ARIGM800GR

ARIBR800GR

ARIKBB350GR

ARIKBB450GR

ARIKBB600GR

ARIKBB700GR

ARIKBB800GR

- WALL MOUNTED ACCESSORIES
- STAINLESS STEEL 304

20250905

## CONGRATULATIONS ON YOUR PURCHASE

Our aim at Better Bathrooms is to bring you extraordinary bathroom products, for less.

If you're after a modern, traditional, or a totally unique style bathroom, we have the fittings and fixtures to bring your ideas to life, so you can fall in love with your space every day.


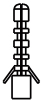
We hope you're happy with your new product. But if you experience any problems, we're always here to help.

- Call our friendly UK call centre on **0330 390 3062**  
(Lines open 9am-4.30pm Monday to Friday, or 9am-2pm Saturday).
- Send us a message via **[betterbathrooms.com](https://www.betterbathrooms.com)**.


## CONTENTS

PARTS SUPPLIED	3
TOOLS REQUIRED	3
INSTALLATION	4
CLEANING AND MAINTENANCE	5


**PARTS SUPPLIED**

	
SCREWS (6)	WALL PLUGS (6)


**TOOLS REQUIRED**

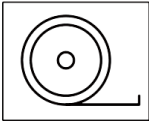


TAPE MEASURE

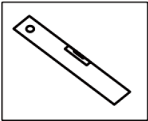


SPIRIT LEVEL

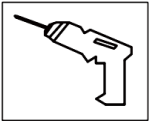


ELECTRIC DRILL

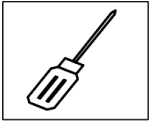


FLAT HEAD  
SCREWDRIVER

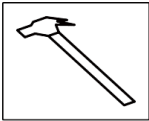

HAMMER

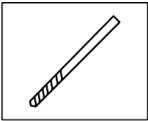

8mm DRILL BIT

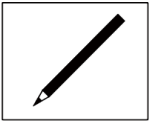

PENCIL

# INSTALLATION


Always check for hidden pipes or cables before drilling to avoid damage.

The fixing plate (attached to the grab rail) must be secured into a suitable structure that is strong enough to support the product and load.

The included screws and wall plugs are designed for solid wall structures such as brick and concrete, for any other wall type use a suitable fitting.


## Remove Decorative Covers from Fixing Plates

The fixing plates are covered by decorative covers. Gently pull or pry off the covers (they are typically clip-on) to expose the fixing screw holes on the plates.

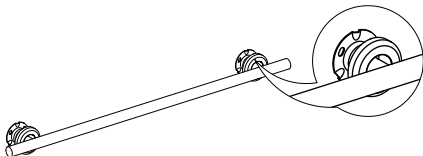


## Select Installation Position & Mark Drill Holes

Choose the desired location for the grab rail. Hold the fixing plates (attached to the grab rail) against the wall. Then, use the fixing plates as a guide to mark the drill-hole positions on the wall.

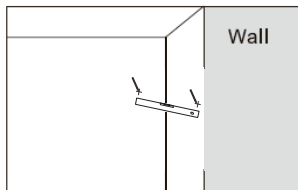


## Drill Holes

Use a suitable drill bit to create holes at the marked positions. If installing on a masonry wall, ensure you have appropriate screws and wall plugs

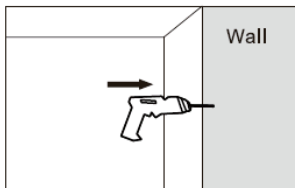

#### Insert Wall Plugs


Place wall plugs into the drilled holes.

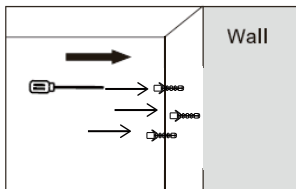

#### Secure the Grab Rail

Align the fixing plates with the wall plugs. Insert screws into the plugs and tighten them fully.

Make sure the grab rail is installed firmly and securely.

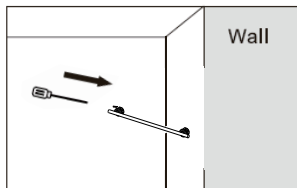

#### Reattach Decorative Covers

Align the decorative covers with the fixing plates and gently press them into place until they clip securely, covering the screws and plates.

## CLEANING AND MAINTENANCE

Never use products containing bleach or abrasive cleaners, as this will harm the product's appearance and lifespan.

Use a clean microfibre cloth to clean the handrail.

#### For more information

- visit [betterbathrooms.com](http://betterbathrooms.com)
- email [support@betterbathrooms.co.uk](mailto:support@betterbathrooms.co.uk)
- call 0330 390 3062
- write to Trident Business Park, Neptune Way, Huddersfield, HD2 1UA